# Sanavox

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Please read the explanatory leaflet carefully before use

## When should it be used?

SANAVOX is a spray for the throat. Its application mechanically protects the mucus of the mouth by hindering contact with dust and other irritating agents and pollutants. It should be used in the case of irritation or for prevention before entering polluted areas or in the presence of irritating agents (cigarette smoke etc.)

## How does it work?

Each application of Sanavox re-establishes the natural reserve of hyaluronic acid that protects our vocal cords, allowing their optimal functioning. Hyaluronic acid forms a natural protective barrier against irritating agents and pollutants whilst keeping the viscoelastic properties of the vocal cords.

## When must it be used subject to medical advice?

If the state of irritation persists despite the use of SANAVOX, you should consult your doctor. Please consult your doctor in the presence of chronic disease of the upper respiratory tract. In the case of the simultaneous use of other nasal sprays containing drugs for the upper respiratory tract, the doctor that prescribed them should be consulted.

## How do you use it?

Insert the nozzle into the mouth and press the dispenser firmly with the index finger and thumb, without bending your head, until you feel the spray reaching the back of the pharynx. Repeat the operation three times.

How often a day can you use it?

Sanavox has a protective effect. In the absence of symptoms, use Sanavox for prevention in polluted environments or in the presence of irritating agents (smoke, dust etc.) At the first signs of rhinitis or a sore throat, use Sanavox regularly, with several applications a day.

Does it have any possible unwanted effects?

No, the application of Sanavox has no unwanted effects.

## Warning

Keep in a cool place at a temperature of not more than 25 °C. Keep away from sunlight. Do not use after the date on the packaging. Keep out of the reach of children. Do not use in pregnancy and when breastfeeding.

## Fabbricante/Manufactured by:

Crab Sinergy srl

Via M. Polo, 18/20

24062 Costa Volpino (Bg)

Tel +39 035.973286

[www.crabsinergy.com](http://www.crabsinergy.com)

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Flacone da 30 ml e

SPRAY orale

Rev\_4b (24/04/2019)